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## GB Causes of problems when using rotary instruments in the dental laboratory

Possible problems with rotary instruments can be divided into three groups.

- ▶ Problems with the rotary instrument
- ▶ Incorrect use
- ▶ Problems with the handpiece

Rotary instruments should be checked regularly.

Use of a magnifying glass is recommended for checking rotary instruments (min. 10 x magnification).

Damaged (e.g. diamond coating worn), bent or non-concentrically running instruments should be discarded immediately.

Remove any dirt from the diamond coating or blades of the instrument. Discs should be checked for any cracks or damage to the steel disc. Working surfaces should be clean when polishing.

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*Check rotary instruments regularly with a magnifying glass (min. 10 x magnification).*

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*The handpiece chuck is defective or dirty and the instrument spins. This damages the shank and the instrument may fracture.*

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*The shank has rust damage. As NTI-Kahla GmbH manufactures the shanks of polishers from stainless steel, rusting may be caused by the effects of acid, e.g. drops when pickling, or contact with non-stainless instruments in a moist environment.*

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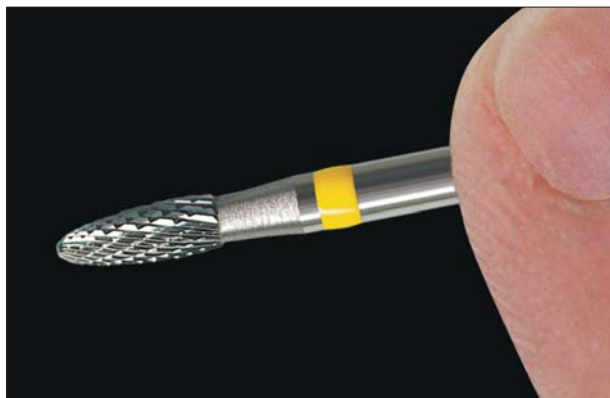
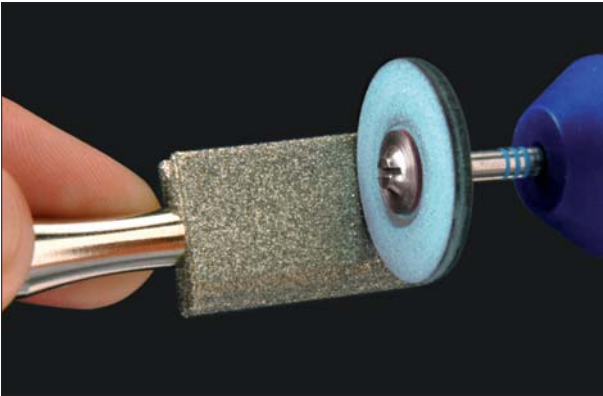
*Clean steel and tungsten carbide instruments with wire brush P6820.*

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*Clean the polishing surfaces with diamond dressing instrument P4060. Polishers should be cleaned and reshaped regularly.*

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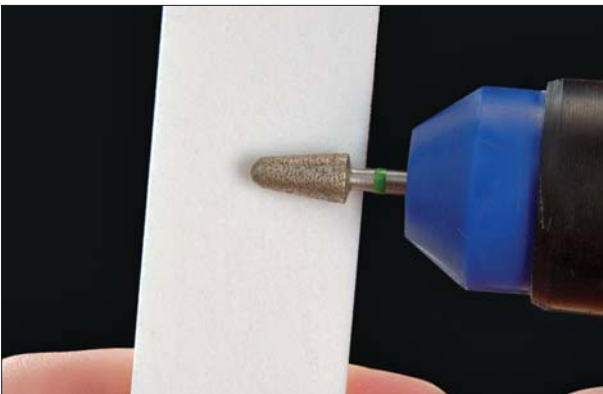
*Dressing Stone P1108 for shaping and pointing of polishers.*

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*Sharpening a sintered diamond on cleaning stone G9920. Application see page 1.29*

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*Cleaning the contaminated diamond coating.*

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## Typical problems when using rotary instruments

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#### Incorrect use

- The motor speed is a crucial factor when using rotary instruments. A basic guideline: the larger the head, the lower the motor speed. Instruments with a diameter greater than 4 mm should not be used at speeds above 20,000 rpm.
- Applying the correct pressure during preparation increases the service life of rotary instruments. Increased pressure does not necessarily improve the cutting performance. If too much pressure is applied, it can have a negative effect on results.

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*Fracture of the polisher has been caused by polishing at too high a motor speed. Large polishers with a diameter greater than 4 mm should not be used at speeds above 20,000 rpm.*

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*Overload fracture of the shank caused by too high a motor speed and chattering of the instrument during preparation. Coarse tungsten carbide cutters with a large head should not be used at speeds above 20,000 rpm or for rapid intermittent preparation.*

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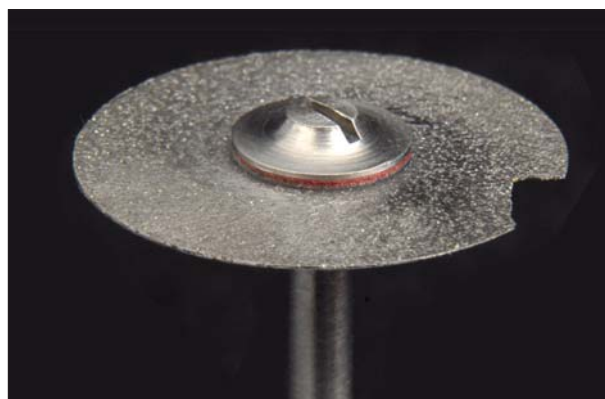
*Broken tip caused by the instrument becoming wedged in narrow spaces, e.g. interdentally.*

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*Fractured section of a sintered diamond disc caused by the instrument becoming wedged, being set down incorrectly or the edge not being cleaned and reactivated.*

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### Typical problems when using rotary instruments

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*Disc destroyed by bent shank. The disc had not been fully inserted into the handpiece chuck and bent because too high a motor speed  $> 25,000\text{ rpm}$  was used during preparation.*

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*The injury of the technician was predictable. The instrument has not been fully inserted into the handpiece chuck and used at too high a motor speed ( $> 20,000\text{ rpm}$ ).*

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*Fractured diamond disc caused by handpiece being set down incorrectly on the work bench.*

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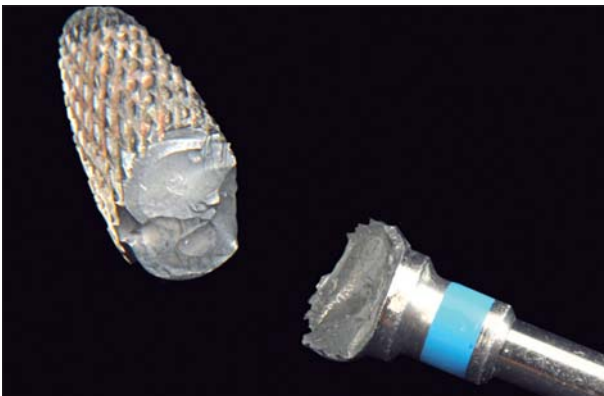
*Fractured polisher caused by handpiece being set down incorrectly on the work bench.*

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*Fractured tungsten carbide cutter caused by handpiece fallen down.*

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### Check and maintenance of dental drives.

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*The hold of rotating instruments in the drive is to be carried out by a manual tensile test on a connected instrument.*

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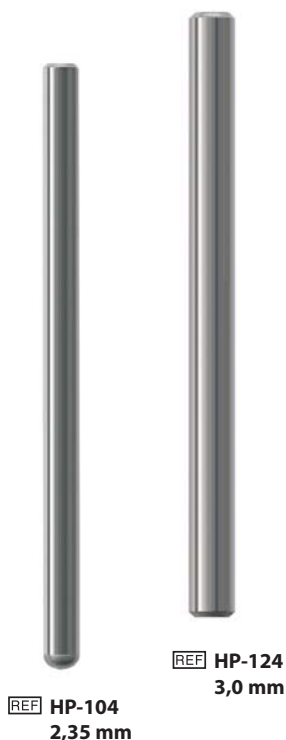
*The drive bearing control can be checked by shaking the instrument. If the instrument does not yield then the bearing is without defect.*

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*High-precision instruments for checking the concentricity of handpieces and milling unit handpieces with 2.35 mm and 3 mm chucks.*

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*Concentricity in the drive:  
It is recommended that the check be done using the appropriate testing instrument.*

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*Cleaning the chuck with a small brush.*

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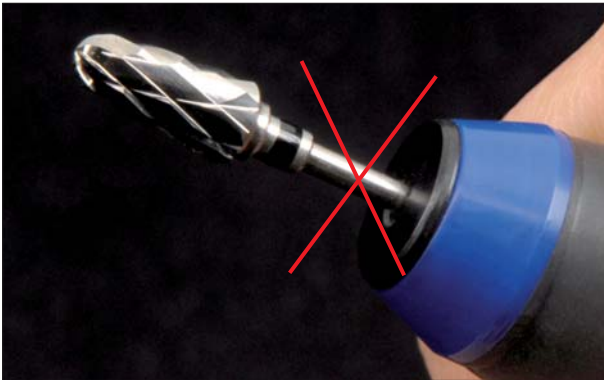
## Check and maintenance of dental drives.

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### Problems with the handpiece:

- *The condition of the chuck has a crucial effect on the service life of rotary instruments. If it is dirty, its retentive force is reduced and the instrument spins. This is not always immediately apparent. When the instrument spins, the shank is damaged and the instrument may fracture. **Caution: Risk of injury !!!***
- *If there is dirt at the bottom of the chuck, the instrument cannot be fully inserted. This can cause the instrument to bend or fracture.*
- *Handpiece chucks are constantly subject to considerable loading. This can easily cause damage, which also results in a loss of retentive force. Chucks should be regularly checked to ensure that they function properly.*

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## Anwendungshinweise • User information • Applications

### GB INSTRUCTIONS FOR USE

NTI-Kahla GmbH instruments have been developed for use in the dental laboratory. Each instrument has been designed for a specific area of application. Use on unsuitable materials or incorrect use can damage the instrument or the object being prepared.

This can also reduce the service life of the instrument. Incorrect use can cause injury or damage the health of the user or a third party.

#### Instructions for use of rotary instrument handpieces:

1. The instruments should only be used in handpieces tested and approved for dental technology purposes.
2. Dental technology handpieces should be regularly cleaned and checked to ensure that they operate perfectly.
3. Handpieces should only be repaired in workshops approved by the drive manufacturer.
4. The rotary instrument should be inserted into the chuck as far as it will go or to the coloured mark.

#### Safety precautions when preparing with rotary instruments:

1. Protective glasses should be worn.
2. Protective clothing, e.g. laboratory coats, prevent injury as well as damage to clothing.
3. Adequate suction of dust when preparing reduces the risk of silicosis.

#### Instructions for use of rotary instruments in the dental laboratory:

##### 1. Choice of instrument:

All rotary instruments can be used without restriction on dental laboratory materials in the dental laboratory.

##### 2. Use of instruments on patients:

The regulations of the Medical Devices Act and the Medical Devices Directive 93/42/EEC and associated laws and regulations apply to the use of the instruments on patients. Instruments that are used on patients have to be approved for that purpose and carry the CE mark:

Designation for :

a) NTI Class 1 products is 

b) NTI Class IIa products is 

Laboratory instruments (without CE mark) are not approved for use on patients.

##### 3. Checking:

The instrument should be checked for damage before use.

##### 4. Practical use:

Set the handpieces to the correct operating speed for the material to be prepared before allowing the instrument to come into contact with the material.

##### 5. Note:

When preparing with a rotary instrument, it is essential to avoid leverage, tilting or excessive pressure.

##### 6. Motor operating speeds:

Adhere strictly to the recommended motor operating speeds for the respective task and instrument.

##### 7. Overheating of the material:

Overheating the material being prepared can have a detrimental effect on the properties of the material. At worst the material can lose the quality required for use in the oral cavity.

##### 8. Overheating of the instruments:

Excessive motor speeds and excessive pressure cause overheating and tempering of the instrument, which can then lose its grinding and cutting capacity and damage the material.

##### 9. Storage:

All rotary instruments should be stored so that they do not hit or rub against one another or come into contact with a base.

Diamond instruments can damage tungsten carbide instruments and tungsten carbide instruments can damage shanks. Polishers that rub against other instruments can contaminate them or cause damage.

All types of discs are easily damaged if stored flat. Diamond and separating discs are damaged to such an extent when stored flat that it can lead to fractures, cracks etc. during use. This may result in injury to the operator.

##### 10. Operating pressure:

Excessive operating pressure increases the risk of instrument fracture. It can also result in damage to the working section, fracturing at the blades and splitting off of the diamond coating as well as increased heat build-up. All this reduces the service life of the instrument. Operating pressure should be 0.2 – 0.5N (20 - 50p). Fine cutting blades or diamonds should be used for fine adjustments. Intermittent contact when reducing the material with super coarse and coarse instruments causes recoil vibrations resulting in shank fracture.

##### 11. Cleaning diamond instruments and metal-bonded sintered diamonds:

Electroplated diamond instruments and sintered diamond instruments with Order Nos. G5009 – G5027; G5102 – G5123; G5206, G5211, G5218; G5331 and G5332 as well as G5113 and G5122 require regular cleaning. Cleaning stone G9920 should be used for cleaning. The cleaning stone should only be used wet. Sintered diamonds will be reactivated and contamination can be cleaned from electroplated diamonds with the cleaning stone.

##### 12. Cleaning tungsten carbide instruments:

The cutting blades of tungsten carbide instruments can be cleaned of surface dirt with a small toothbrush and in the case of ingrained dirt with the wire brush P6820. The instrument should never be heated.

##### 13. Cleaning polishers:

All polishers in the NTI range can be cleaned and reshaped with the diamond dressing stone P4060.

Dress and adjust un-mounted polishers before the first use.

##### 14. Special precautions when using chemical fluids:

Instruments should not come into contact with acids, caustic solutions, methylmethacrylates (monomers), alcohol or H<sub>2</sub>O<sub>2</sub> (hydrogen peroxide), as this could cause irreversible damage. Extra care should be taken when pickling alloys at the workbench.

##### 15. Steel instruments:

Steel instruments made from unconditioned tool steel are very sensitive and should be protected particularly against any kind of liquid, including water, and stored dry.

Any liquid spilled on an instrument should be removed immediately.

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