# Kronentrenner Metall

## Metal Crown Cutter

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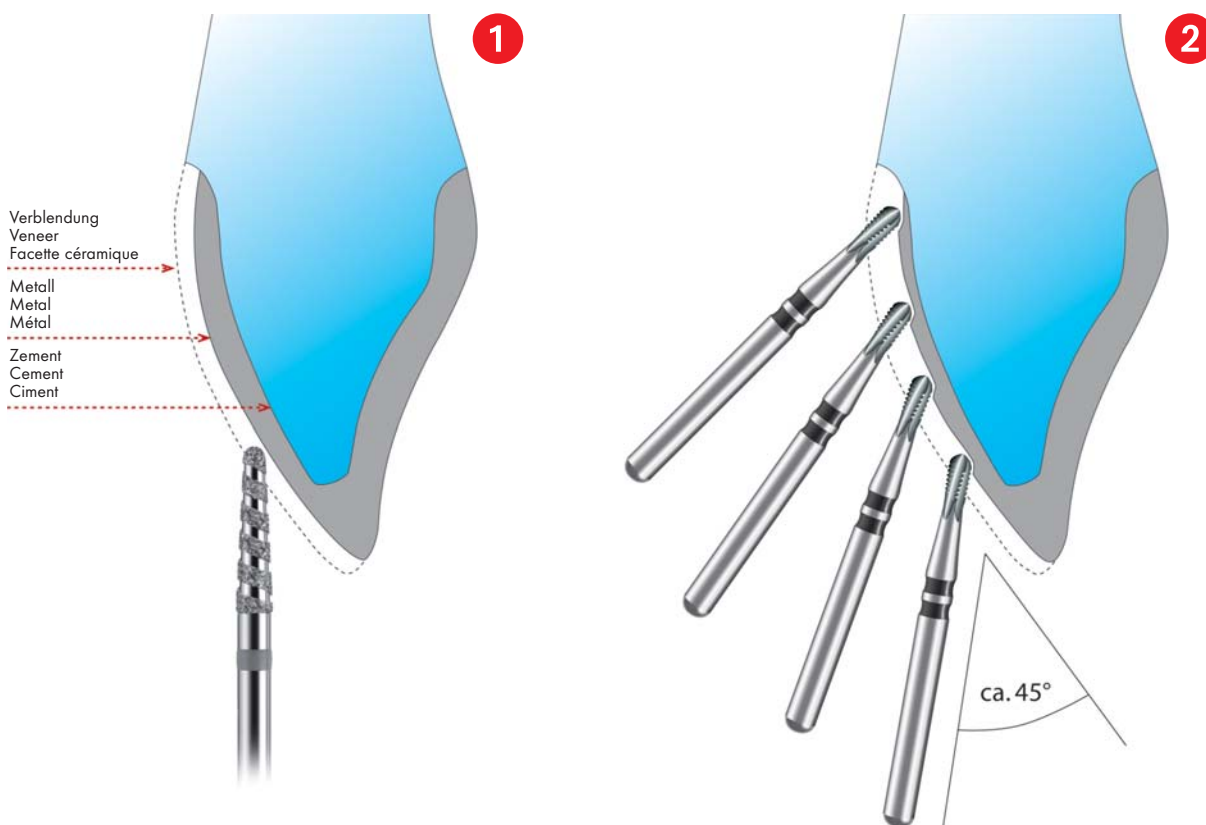
#### Features and Applications:

- for removal of crown and bridge restorations
- optimal geometry of toothing
- high material reduction
- vibration-free because of one-piece-construction
- longterm performance

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### When cutting crowns and bridges the following recommendations should be observed:

- 1 The veneer should be cut with a diamond instrument first.
- 2 Reduce the crown metal layer gradually, which means starting the procedure from top to bottom end several times.  
opt. speed 160,000 r.p.m.,  
Contact pressure: 0.5 N  
Please observe the working angle of 45 degrees!

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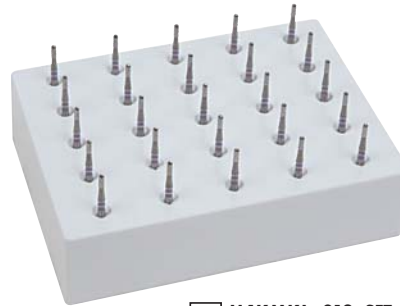
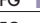
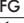
#### Features and advantages:

- Used for alloys faced with low-fusing porcelain
- The special blade design ensures high cutting capacity.
- The tip is designed for cutting and slices into the crown.
- One-piece production ensures high stability and reduces the risk of fracture.
- One instrument for two procedures - no need to change between diamond and carbide instruments.

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### Sintered diamond disc for separating crowns

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Caution: use spray coolant when preparing.

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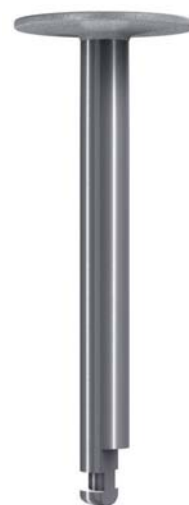
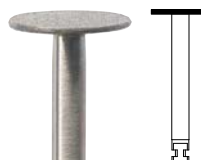
Crowns quickly separated through any material including zirconium oxide

- The 127  $\mu$ m grit size has a high cutting capacity that ensures all types of crowns can be quickly separated.
- The diamond grit cuts through gold, porcelain, non-precious metals and even zirconium oxide using one instrument.
- The disc facilitates separating bridges intraorally.
- With continuous cooling it swims over the surface of the gingiva, reducing the risk of damaging the gingival margin.

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**Important:** It is essential to cut a deep "V" into the crown. The cut is wider at the surface than deeper into the crown. Otherwise the disc can become stuck between the two sections of the crown, bend and/or break if the patient moves. Not suitable for the lifting of crowns (risk of fracture)!

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