REF Set-1642 

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Preparation Set for CEREC® Veneers designed by Dr. Klaus Wiedhahn

Set for preparing CEREC® veneers and final finishing developed in collaboration with Dr. Klaus Wiedhahn.

The coordinated instruments for preparing veneers guarantee optimum preparation and consequently accurately fitting restorations.

The coarse grit size of the preparation diamonds produces rapid, time-saving reduction of the tooth structure.

The fine instruments create optimum preparation margins, which are required for accurate recording with an intraoral camera and facilitating digital data processing. This guarantees a gap-free fit of the veneer at the natural tooth.

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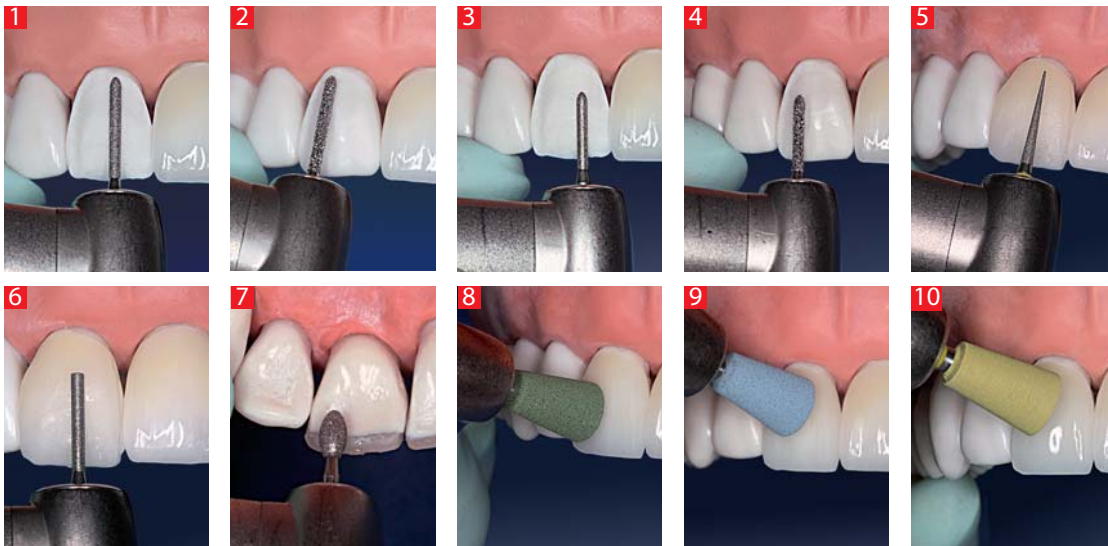
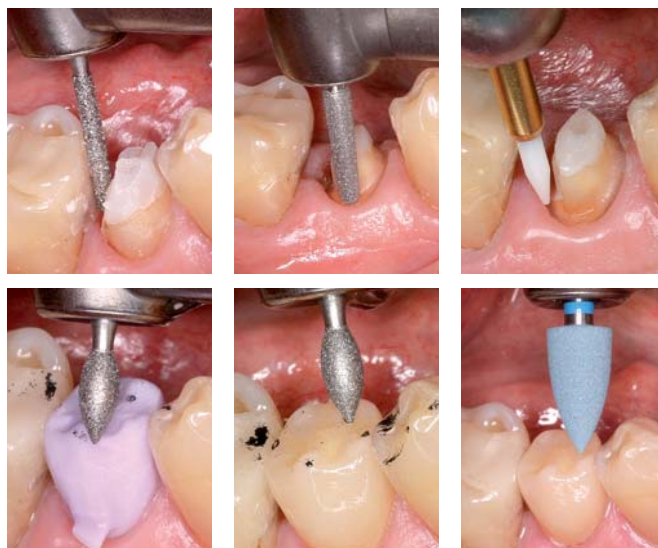
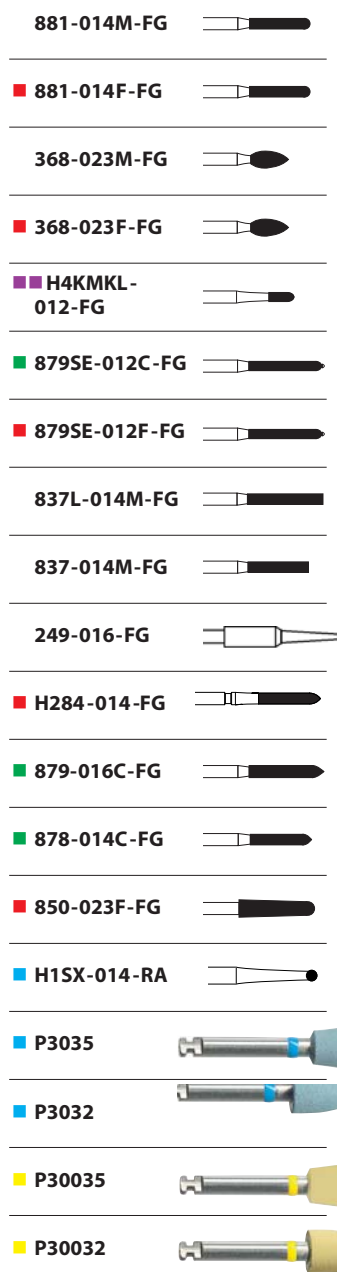
An instrument set for perfect CEREC® crown and bridge preparation designed by Dr Klaus Wiedhahn.

The set contains all the essential instruments for crown and bridge preparation, from removing old crowns to safe exposure of the cervical preparation margin using the tissue trimmer.

The set includes coarse and fine preparation instruments. These facilitate rapid preparation for crowns and bridges fabricated using CAD/CAM.

The low number of instruments simplifies the preparation sequence.

The CeraGlaze polishers in the set create the same surface texture as that produced following standard glaze firing, without any risk to the marginal seal of the restoration or delayed crack formation.

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ALL IN ONE All-Porcelain Set designed by Piet Troost

With this compact, high quality set, every type of all-porcelain restoration preparation is possible. The most recent preparation requirements for modern adhesive dentistry, including CAD/CAM technology, have also been accounted for.

This selection of instruments facilitates effective and time saving preparation for all requirements of all-porcelain restorations, with minimal bur exchange: from all-porcelain inlays, onlays and partial crowns to crowns and bridges with a modified chamfer (CAD/CAM).

In addition to conventional labial veneers (porcelain veneers), the ALL IN ONE set is also shaped optimally for palatal, occlusal and 360° veneers.

The ALL IN ONE all-porcelain set provides the dentist and his team with a comprehensive solution for the production of high quality, metal-free restorations.

Time-saving and effective, this set is well equipped for every type of all-porcelain challenge.

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Axial Reduction Instruments by Dr Mark J. Caldwell

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MADC Depth Markers by Dr Mark J. Caldwell, USA

Depth marking for all areas of crown and bridgework.

All-porcelain and CAD/CAM produced restorations place high demands on dental preparations. The success of a restoration depends to a high extent upon the compliance of the minimal wall thickness of the restoration (Teamwork 6/2007; Prof. Dr. Frankenberger et.al.).

With the „Caldwell“ (acc. to Dr Mark J. Caldwell) diamond instruments, these requirements can be safely and easily achieved.

With these four instruments it is possible to define the prepared margin at the depths of 0.6 mm; 1.0 mm; 1.5 mm and 2.0 mm. The MADC instruments have ring markings stating each individual depth. This way you can see the differences even when in the bur stand. With these instruments all preparations are possible, with any technically or physically required depth.

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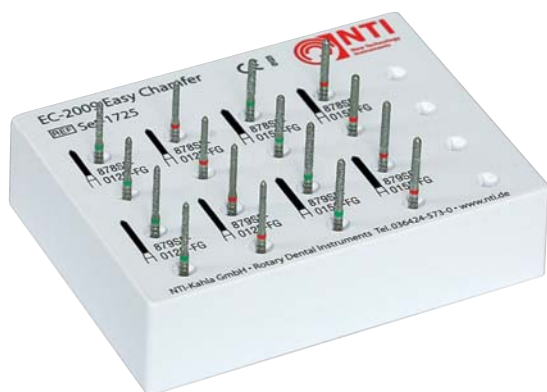
■ 879SE-015C-FG 2x 

■ 879SE-015F-FG 2x 

Optimal deep chamfer preparations - fast - easy - perfect.

The abrasive-free tip facilitates contouring the cervical margin and produces an optimal deep chamfer, which meets the requirements for all-porcelain restorations.

There is now another size available of the successful 879SE instrument, which was introduced at the IDS 2007.

The additional diameter of 015 facilitates circumferential preparation of premolars and molars. The larger instrument reduces the time required for preparation.

The two longer instruments are complemented by the shorter torpedo shape 878 with diameters of 012 and 015.

This allows optimal deep chamfer preparation, even in the dorsal region of the molars and with restricted mouth opening.

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Prophylaxis Set 1 Set-1643

A complete set for cleaning and polishing teeth, amalgam and composite fillings and precious metal alloys.

The snap-on polishers are used with polishing paste for polishing teeth. The reusable mandrel makes the polishers highly cost-effective.

Abrasive material is integrated in the blue prophylactic polishers. Polishing paste is not therefore required. With very stubborn deposits, polishing paste can be used to increase the polishing capacity.

The small nylon brushes are used with polishing paste. They are ideal for use in the occlusal area and interdental spaces.

The NTI EpsiPol is suitable for use on all precious and semi-precious metal alloys as well as amalgams. The brown prepolisher removes scratches and the green polisher produces a perfect lustre.

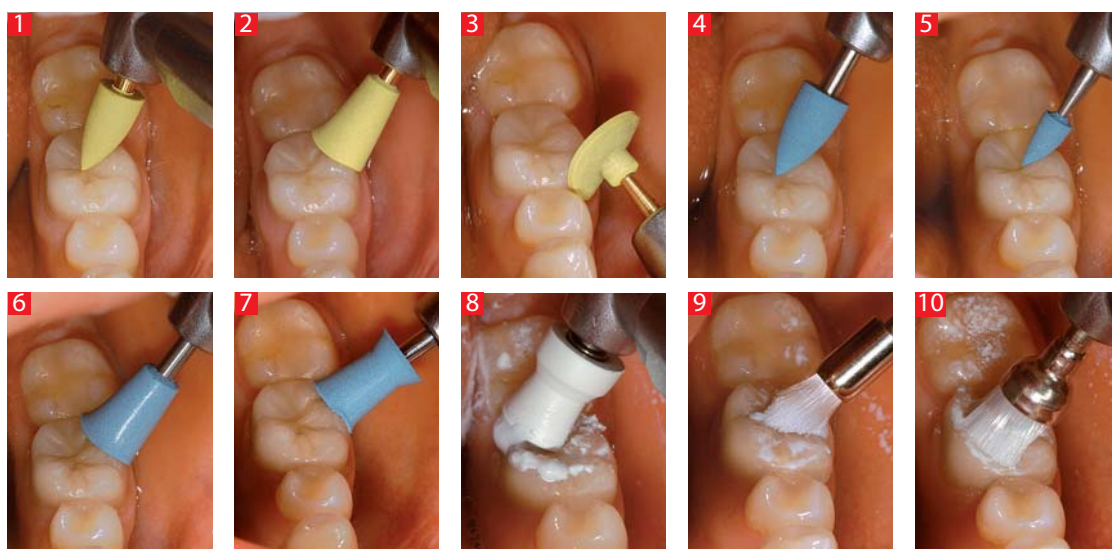
The yellow Unique polisher is a diamond-coated one-step polisher for composite fillings. The selected diamond grit allows all types of composite to be polished with a single polisher. This reduces costs and the time required for polishing.

The newly developed silicone binder, which combines flexibility with stability, can be used in all areas and prevents damage to the composite surface.

Prophylaxis Set 2 Set-1644

This prophylactic set is a complete set just for cleaning teeth.

This combination of snap-on polisher, blue prophylactic polisher and small nylon brush facilitates polishing. Different shapes allow use on any type of tooth.

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Universal Composite Set

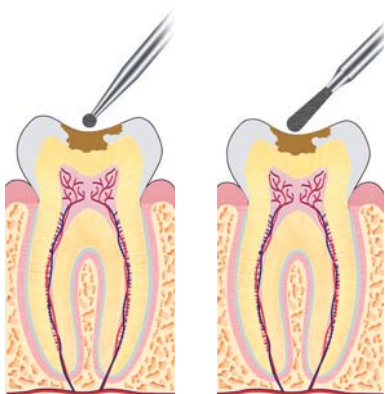
includes all necessary rotary instruments for Caries elimination, Removal of old fillings, Cavity preparation classes I, II, III, IV and V, Contouring, Finishing and Polishing of Composite fillings.

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Removing decay dentine. Opening and widening the cavity.

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Removing old amalgam or composite fillings with carbide burs.

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
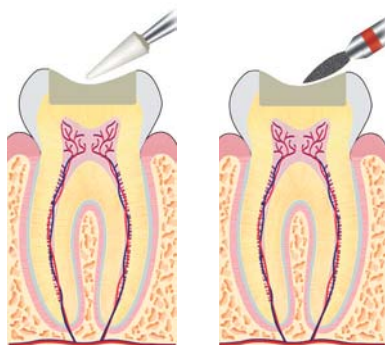
After hardening the surface of the composite filling is trimmed and contoured.

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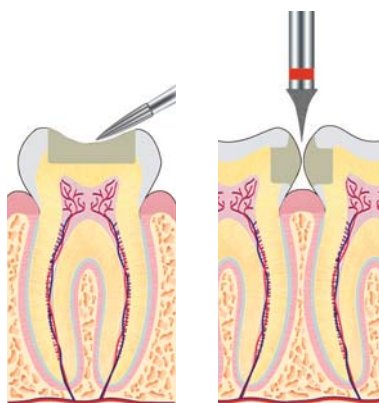Finishing the surface with finishing carbide burs.

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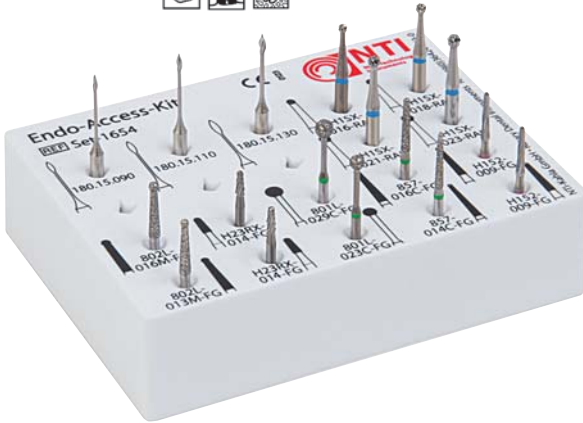
Polishing with NTI Unique, diamond impregnated one stage polishers, for best results use with water! Abrasive polishing at a speed of 15,000 rpm; glaze polishing at 10,000 rpm; work with slight pressure.

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Endo Access Kit

Well-structured instrument set for quick, safe opening of the pulp chamber prior to endodontic treatment.

The instrument set facilitates precise preparation for endodontic treatment with any indication. The instruments can be used for preparing the natural tooth crown as well as prosthetic crowns. As the instruments are available in varying sizes, they can be used for any type of tooth.

Instruments with non-cutting ends prevent excessive expansion of root canals and avoid perforation.

Gates drills facilitate preparation in narrow physiological areas.

The tungsten carbide instruments are manufactured in one piece. This reduces the risk of fracture and ensures perfect concentricity.

The special cutting blades of the H23RX-014-FG produce rapid reduction so that preparation of large areas of metal crowns requires less time.

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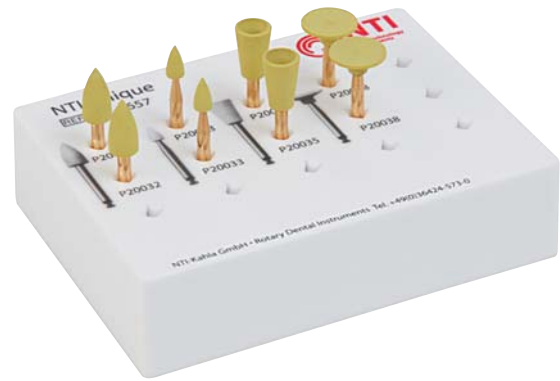
Composite Finishing Kit

This assortment is for finishing and polishing all microfilled composites. Produces a high lustre quickly.

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NTI Unique Set

The selected diamond grit enables polishing of all types of composite using one polisher, this reduces costs and saves time.

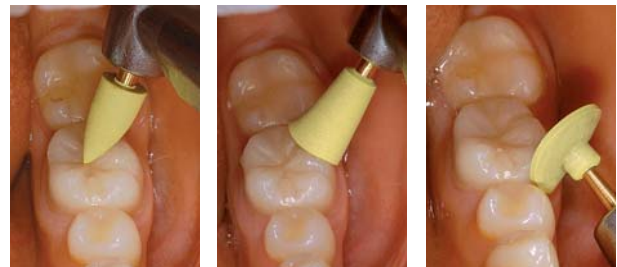
The newly developed silicone bonder with elastic stability enables polishing in all areas and prevents damage to the composite surface. The continuous release of abrasive materials produces a smooth and high-lustre surface.

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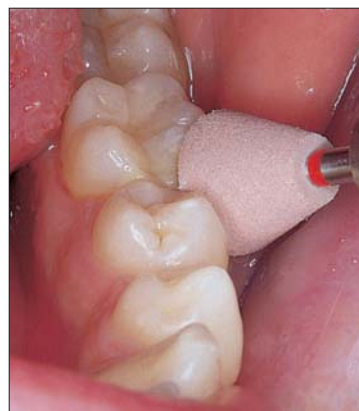
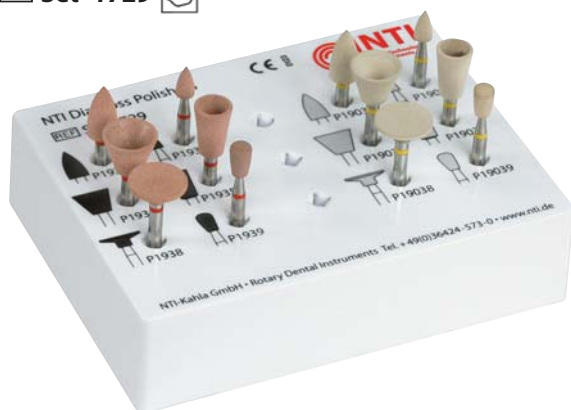
NTI DiaGloss Set

First class pre-polishing and high gloss polishing of composites in two steps. The pink DiaGloss pre-polisher with a fine diamond grit smooths the surface and removes last scratches.

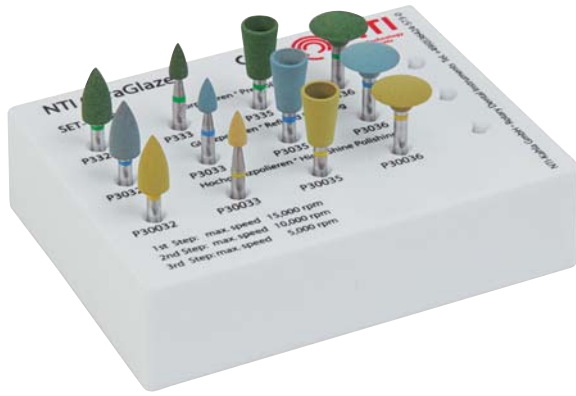
The white DiaGloss with a superfine diamond grit provides a perfect high-gloss polish on composites. For preparation of composite surfaces we recommend fine and superfine diamond instruments or Finishing Tungsten Carbide.

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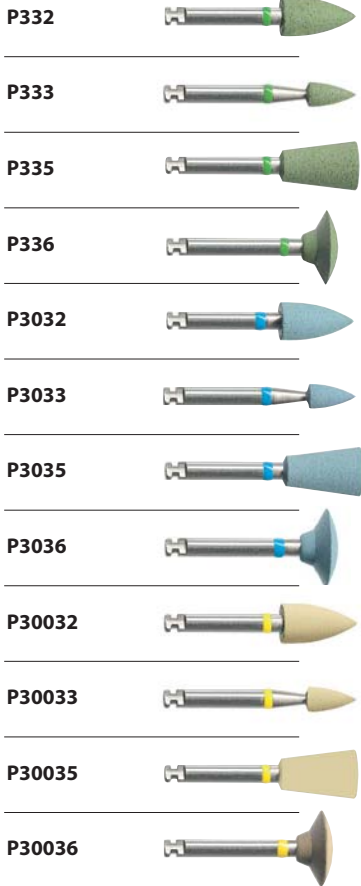
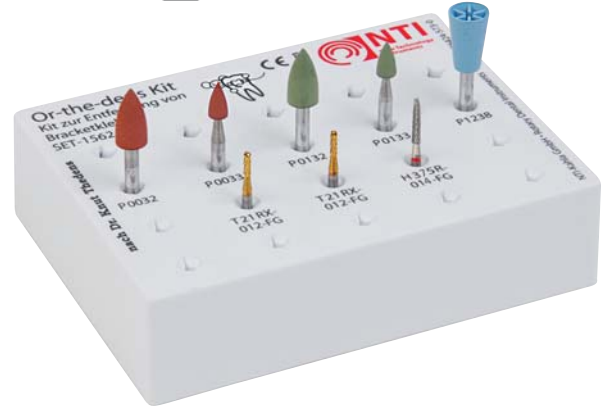
NTI CeraGlaze RA

Porcelain diamond polishers, tried and tested for years in laboratory use, now also available for preparing and polishing intraorally.

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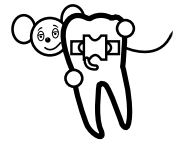
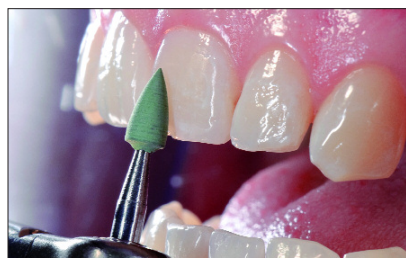
Or-the-dens Kit


Set for removal of bracket bonding resin designed by Dr Knut Thedens.

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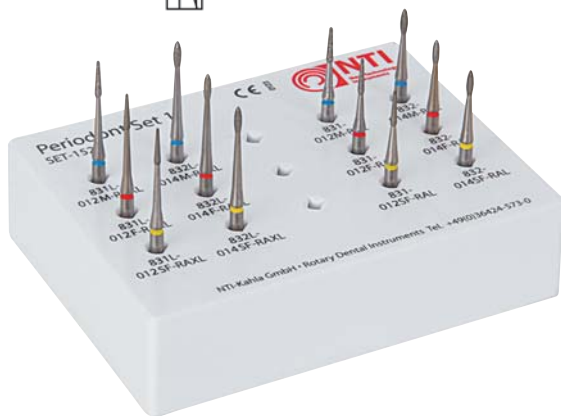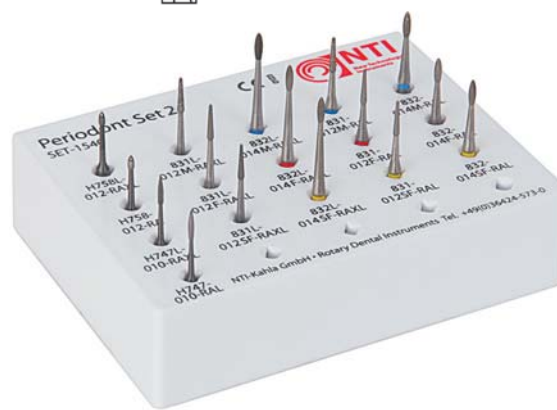
Periodont Set 1

(Periodontal Diamond Set) Two different shapes with two different shank lengths for trimming root surfaces.

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Periodont Set 2

(Periodontal Diamond and Tungsten Carbide Set)
Two different shapes with two different shank lengths, one each tungsten carbide and diamond, for trimming root surfaces.

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Z-Cut diamond instruments

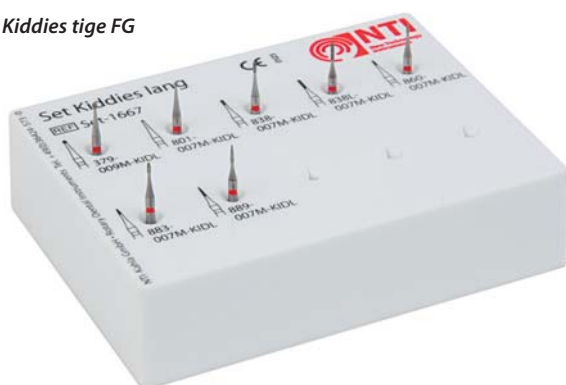
Ideal for the preparation of zirconia abutments and separation of zirconia crowns.

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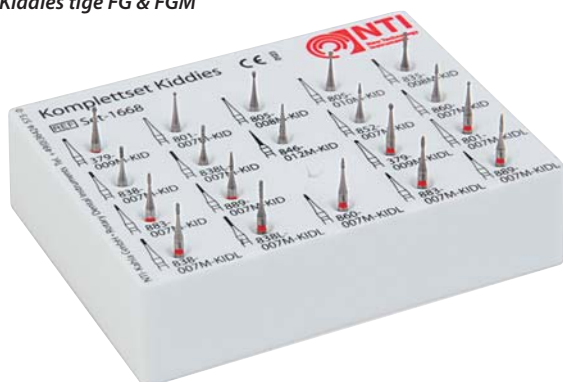
Description Sets Kiddies instruments on page 1.30

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